




Technologies Freeform® OCCUPATION IntelliCorridor™ EPT EyePoint Technology™	Product Classification: WORK > Progressive	Marking:
	Minimum Fitting Heights: 16mm lower & 10mm upper	FFO

The Lens Design

As you can see in the diagram below, with Shamir **Autograph II - Office™** we are able to enhance the patient's experience by providing them a personalized Dynamic Power based on the additional measurements taken. Shamir **Autograph II - Office™** caters to every detail and aspect of the patient's work day, ensuring that they experience the most refined, personalized vision. What's even better than a great day at work? A great day at work with a pair of Shamir **Autograph II - Office™** lenses.

Shamir Autograph II – Office™

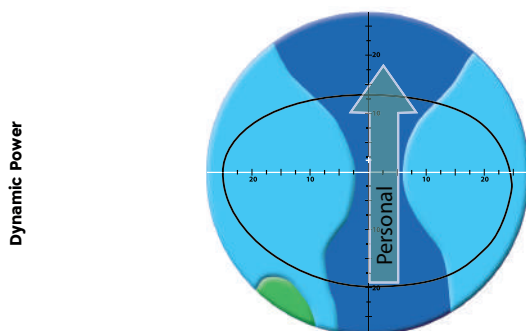

One personal Dynamic Power calculated to the patient prescription, frame and personal space.

Who It's For

Anyone wearing progressive lenses that spends the majority of their time at their workstation or enjoying hobbies like woodworking and sewing.

Features & Benefits

- Provides a personalized Dynamic Power
- Incorporates patient's work environment
- Designed with EyePoint Technology®
- Ideal for work or hobbies
- Personalized to fit the patient and their work station

Occupations/Hobbies that may benefit from Shamir **Autograph II Office™** are:

- Physician
- Hair Dresser
- Jeweler
- Tailor
- Accountant
- Cashier
- Graphic Designer
- Chemist
- Woodworking
- Sewing
- Homebrewing
- Gardening
- Cross Stitching
- Board Gaming



IntelliCorridor™

Unique Power Profile For Clearer Vision

Makes it possible to customize lenses specifically to the patient's vision requirements. Optimizes the power in specific areas used for everyday and outdoor tasks, reducing the need to move head up and down.




EyePoint Technology™

The Design Inside

EyePoint Technology® simulates the human eye in every angle, prescription, and field of vision. These techniques enable Shamir to create the most sophisticated progressive lens surfaces based upon thousands of points of data. It's this "design inside" that makes Shamir progressive lenses the most advanced in the world.

Technical Information

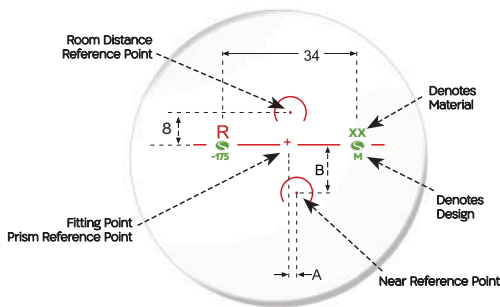

Fitting Height	A (see diagram)	B (see diagram)	M (variable fitting see diagram)
16mm	25	12	FFO

Verify Lens at NEAR REFERENCE POINT

How To Fit & Order Shamir Autograph II Office™

Provide the following information:

- Patient Distance and Near Rx do not adjust
- Fitting Height
- Monocular Distance PD
- Frame Measurements A/B/DBL/ED

Minimum Design Fitting Heights

Lower: 16 mm

Upper: 10 mm

(min. distance from fitting point to upper frame rim)

THE FULL DYNAMIC REDUCTION IS ACHIEVED AT 8 MM ABOVE THE FITTING POINT.
VERIFY AT THE NEAR.

Availability:

Material	Total Near Power	Addition	Cylinder
1.50 Hard Resin (C)	-6.00 to +6.00	0.75 to 3.50	to -6.0 [D]
1.6 SuperLite™ (C)	-6.00 to +8.00	0.75 to 3.50	to -7.0 [D]
Polycarbonate (C)	-6.00 to +8.00	0.75 to 3.50	to -7.0 [D]

LEGEND:

- C - Clear
- TV - Transitions® Vantage™
- T - Transitions®
- P - Polarized
- TX - Transitions® XTRActive™
- B - BluTech (Indoor & Outdoor)
- D - Drivewear®
- N - NXT™
- BZ - Blue Zero™
- S - SunSync®

POWER RANGE IS DETERMINED BY SHAMIR CERTIFIED BLANK RANGE PRISM TO 10 DIOPTERS IN EACH QUADRANT

⚠ WARNING: Polycarbonate lenses can expose you to chemicals including Bisphenol A (BPA) and 1.74 Index lenses can expose you to Methimazole, which are known to the State of California to cause birth defects or other reproductive harm. For more information go to: www.P65Warnings.ca.gov.



/ShamirInsight



@Shamir_Insight



thevitaminsee.com



SHAMIRLENS.COM 877.514.8330

TECHNICAL INQUIRIES 888.370.0736